

MASK <small>frame, cushion & headgear</small>	TUBING	HUMIDIFIER	MACHINE	FILTER
<p>DAILY</p> <ul style="list-style-type: none"> Wipe cushion with PAP Mask Wipes <small>(warm soapy water can also be used)</small> Air dry during day <small>(avoid direct sunlight)</small> <p>WEEKLY</p> <ul style="list-style-type: none"> Hand wash all parts with Mask & Hose soap <small>(warm soapy water can also be used)</small> Air dry during day <small>(avoid direct sunlight)</small> 	<p>WEEKLY</p> <ul style="list-style-type: none"> Hand wash with Mask & Hose Soap <small>(warm soapy water can also be used)</small> Air dry during day <small>(avoid direct sunlight)</small> <p>MONTHLY</p> <ul style="list-style-type: none"> Check for holes and damage 	<p>DAILY</p> <ul style="list-style-type: none"> Remove humidifier Empty remaining water Rinse with warm water Air dry during day <small>(avoid direct sunlight)</small> <p>WEEKLY</p> <ul style="list-style-type: none"> Dismantle humidifier Soak 30 mins in 1 part vinegar and 5 parts warm water Rinse thoroughly before next use 	<p>WEEKLY</p> <ul style="list-style-type: none"> Wipe with PAP Mask Wipes <p>1-2 YEARS</p> <ul style="list-style-type: none"> Service machine <p>WARNING Usage of ozone to clean PAP machines can void the CPAP machine warranty</p>	<p>WEEKLY</p> <ul style="list-style-type: none"> Check for dust build up Ensure filter is not rotated when returned to machine <small>(can suck debris into motor)</small> <p>1-6 MONTHS</p> <ul style="list-style-type: none"> Replace filters as needed

Bundle & save 10% OFF



Wipes



Mask & Hose Soap



Tubing Brush



ResMed S10/S9 Filters



Philips DreamStation Filter



F&P SleepStyle Filters